Importing images into PACS from Powershare

https://www1.nuancepowershare.com/



2. Ensure the Upload To dropdown displays MARY WASHINGTON HEALTHCARE. Click on Choose Folder

Upload To	MARY WASHINGTON HEALTHCARE					
Choose Fol	der ☐ Scan Medical Images Only					

- 3. Select the DVD drive or folder on our computer and click Upload. This is typically your D: or E: drive if using a CD/DVDd.
- 4. When the patient information populates, click Edit on the right side

1 Medical Images Selected		elected	No Other Images Available						
√≎	Name	DOB	MRN	DICOM Study Date	Modality	Body Part	Images	Tags	
	TESTPATIENT	05-26-1941	821359	06-03-2019	CR		3	Select Tags	Edit

Continued on Next page...

5. Modify the MRN to match the MWHC MRN and remove the Accession Number to make the field blank. Once complete, click Save

Edit
. .

- 6. Click Save and then Upload. Wait until you see the Upload Complete message before leaving the page. If you leave this page prior to the Upload Complete message, the images will not import to Powershare.
- 7. Find the study on the Images tab and select the study on the left side and click Push to Accelerator.

	Name T	Upload Date (ET)	DICOM Study Date	Modality T	Study Status All V	Source	Dicom Inst.	 Viewer Option Basic Clinical
✓	D TESTPATIENT PACS 5/26/1941	10/18/2021 9:58am	6/03/2019 8:52pm	CR		MARY WASHINGTON HEA	MEDICAL IMAGING OF	1 Study Selected Show Unselect All Actions
								Unshare these Images Unshare these Images View Share History Download DICOM Nominate to PACS Push to Accelerator
								Copy Delete Studies

8. For Radiology Intellispace (iSite) PACS, choose STENTOR_SCP (Mary Washington Philips PACS) For Cardiology Intellispace (ISCV or Xcelera) PACS, choose XCELERASCP (MWHC XCELERA on MaryWash new) Once selected, click Send to Accelerator



9. If pushing to Intellispace Radiology PACS, login and resolve Exception as normal