

Enterprise Imaging Keyboard Shortcuts

Explanation of shortcut	Keyboard shortcut
Zoom the image	Ctrl + Mouse scroll wheel
Pan an image	Ctrl + Left mouse hold and drag
To adjust the linked position of a single series	Shift + Mouse scroll wheel
To adjust the linked position of a study	Shift + Ctrl + Mouse scroll wheel
To change the time step number in a series that has multiple time steps	Alt + Mouse scroll wheel
To advance a single image	PgDn
To go back a single image	PgUp
To start dictation on a study	Ctrl + Enter
Use the “Active Target” tool when two or more different planes are displayed	Click and hold the mouse scroll wheel