



Keyboard Shortcuts for XERO® Viewer.

Explanation of shortcut	Keyboard shortcut
Reset ONLY the brightness and contrast without resetting other modifications	Shift + Left mouse Click
Pan an image	CTRL+ Left mouse hold and drag
Reset image zoom or pan without resetting other modifications	CTRL+ Left mouse Click
Fast navigation a group of images	Shift + Mouse scroll wheel
Toggle display of an image between a 1x1 layout and the previously selected layout	Double click left mouse
View XERO Viewer in full screen mode or exit XERO Viewer full screen mode	F11
To navigate images one at a time	Next image – Right arrow Prior image – Left arrow
To navigate ten images one at a time	Next image – Right arrow Prior image – Left arrow
To move to the first or last image in the stack	Last image – End First image - Home
Zoom in and out	Zoom in - +Button Zoom out - -Button